



SPECIALTY LABORATORIES

2211 Michigan Avenue
Santa Monica, CA 90404

800•421•4449

Specialty's HIV-1 RNA UltraQuant[®],
Hepatitis B Virus DNA UltraQuant[®]
Hepatitis C Virus RNA UltraQuant[®]
Are ***Ultra Convenient.***



Specialty's new **One-Step Draw & Delivery** system allows you to collect specimens for HIV viral load testing without aliquoting plasma and without shipping frozen on dry ice. With the One-Step system you simply package and ship specimens to *Specialty* at ambient temperature.*

Here's what you do:

Step 1: Draw 5 mL Whole blood in each of 2 PPT™ tubes

Step 2: Centrifuge at room temperature for 10 minutes at 1,100xg RCF within 2 hours after collection

Step 3: Put tubes and requisition in *Specialty* specimen bag

Step 4: Call for pick-up (800) 421-4449 or ship by overnight courier

It's that simple.

- No plasma transfer or relabeling
- No freezing*
- No packing on dry ice in bulky containers

*Specimen can be frozen after centrifugation and shipped frozen.